ENGLISH	2
DEUTSCH	4
NEDERLANDS	6
FRANÇAIS	8
SVENSKA	10
DANSK	12
SUOMI	14
POLSKI	16
ITALIANO	18
ESPAÑOL	20
ΕΛΛΗΝΙΚΑ	22
PORTUGUÊS	24
SRPSKI	26
HRVATSKI	28
TÜRKÇE	30
SYMBOLS	32

# Innovamix®

## Bone Cement Mixing and Dispensing System

Innovamix® V (LB-JV039, GTIN: 8719327432709)

Innovamix® U (LB-JU039, GTIN: 8719327432716)

**MD**

Leader Biomedical Europe B.V.  
Hessenbergweg 109, 1101 BS Amsterdam, The Netherlands  
T: +31 20 800 6760 | [www.leaderbiomedical.com](http://www.leaderbiomedical.com)

# Innovamix®

## Bone Cement Mixing and Dispensing System

### INSTRUCTION MANUAL

**CAUTION:** Federal law (U.S.A.) restricts this device to the sale by or on the order of a physician.

#### DESCRIPTION

The Innovamix® bone cement mixing and dispensing system is a sterile and fully disposable single-use device designed for vacuum mixing and delivery of acrylic bone cement. It consists of an integrated mixer and extruder device (gun), base, funnel, nozzle and a vacuum tubing set.

The Innovamix® bone cement mixing and dispensing system is provided in two configurations: Innovamix® V with short nozzle for procedures of cementation of both the tibial and femoral implantation sites and Innovamix® U with long nozzle and femoral pressurizer for procedures of cementation of the femoral canal. It is recommended that a combination of Innovamix® V and Innovamix® U is used for procedures of cementation of the acetabulum and femoral canal.

The Innovamix® system is used to mix up to 80 grams of cement (including monomer).

The recommended vacuum range to operate the device is  $\geq 100\text{mBar}$  and  $\leq 500\text{mBar}$ .

#### CAUTIONS

Always read the instructions in full prior to using the product. Do not proceed if there is anything unclear. The user should be familiar with the system and the cement prior to use.

Before using the Innovamix® system, the user should have read instructions for use of the selected bone cement and confirmed that bone cement is compatible with the Innovamix® system.

#### STERILITY


The Innovamix® system is provided sterile by prior exposure to EtO-Sterilization. Sterile only if package is unopened and undamaged. Always inspect pouches for damage or holes before using the product. Do not use if package is opened or damage. Do not resterilize.

### INSTRUCTIONS FOR USE (Refer to pictures last page)

#### CAUTION

Inspect package for damage.  
Do not use if package is opened or damaged.  
Read all instructions prior to use.

#### INNOVAMIX® - COMPONENTS

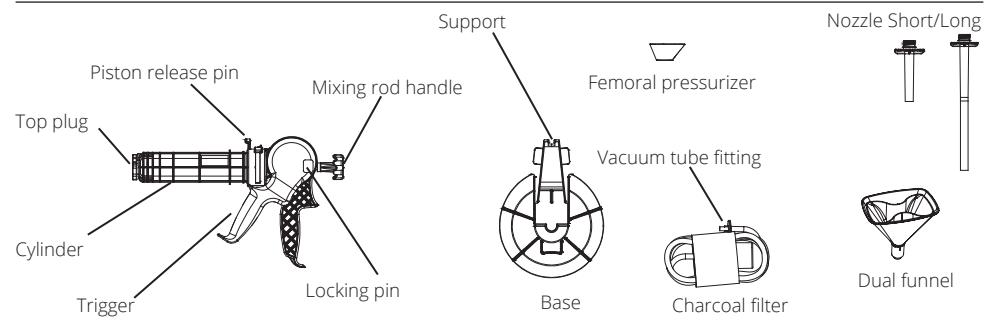#### Step 1: Start - Position

Pull out the mixing rod handle 5cm–7cm. Set the mixing rod handle into the base and lock into position using the support. Unscrew the top plug and set aside for later use.  
Caution: Make sure that the base rests on a flat and steady surface.

#### Step 2: Funnel - Vacuum Tubing

Put the dual funnel into place. Connect the vacuum tube fitting to the top of the cylinder.  
Caution: Make sure that the funnel is securely seated into the cylinder top.

#### Step 3: Monomer - Filling

Start the vacuum pump. Fill the monomer.  
Caution: Use vacuum during filling monomer. This significantly reduces exposure to monomer fumes.

#### Step 4: Powder - Filling

Detach the inner funnel. Fill the cylinder with bone cement powder.

#### Step 5: Powder - Filling

Detach and dispose the funnel. Thread the top plug into cylinder top until tight. Caution: Ensure the top plug is tight.

#### Step 6: Mixing - Start

Detach Innovamix® from the base. Using the mixing rod handle, move the mixing rod back and forth with rotating action.  
Caution: Recommendation for mixing duration; 30–40 sec.

#### Step 7: Nozzle - Select

Stop the vacuum pump and remove the vacuum tube fitting. Unscrew and dispose the top plug.

#### Step 8: Nozzle - Select

Select nozzle and screw firmly in place.

#### Step 9: Dispense Mode - Engage

Pull the mixing rod handle to its rearmost position. Remove and dispose the piston release pin.

#### Step 10: Dispense Mode - Engage

While holding the Innovamix®, rotate the cylinder (either direction) to a full stop.

#### Step 11: Dispensing - Ready

Remove and dispose the locking pin.  
Caution: Make sure that the mixing rod handle is parked at its rearmost position before removing the locking pin.

#### Step 12: Dispensing - Ready

The gun is now ready for dispensing.  
Caution: The bone cement must be extruded well within the working phase period. Injection of bone cement at or near set time may cause cylinder to fracture.

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